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INSTRUCTIONS FOR USE

MODE D'EMPLOI

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INŠTRUKCIA

BRUGSANVISNING

INSTRUKTIONER

BRUGSANVISNING

KÄYTTÖOJEET

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**VENOSAN® Anti-embolism stockings (AES)****Indications**

- Pre/post-operative thrombosis and embolism prevention
- General immobility, confinement to bed
- Before, during and after birth

**Contraindications**

- Dermatitis
- Phlebitis
- Severe arterial sclerosis
- Ischaemic vascular disease
- Dropsy of the legs, pulmonary oedema with congestive cardiac failure

Long, post-operative confinement to bed or general immobility causes a severe slowing down of the venous return, and clots may form as a result. If these clots become detached from the vein, they can lead to pulmonary embolism.

If they remain in the vein, however, they will lead to damage, causing venous insufficiency and varicose veins. Chronic venous insufficiency (CVI) is the cause of *ulcus cruris*.

VENOSAN® AES anti-embolism stockings increase the flow rate of the venous return and help to prevent the clots that lead to pulmonary embolism or chronic venous insufficiency.

**Material composition:**

AD to knee

AG to thigh

AGG to hip with belt

Contain no latex

AD 91 % polyamide, 9 % elastane

AG 91 % polyamide, 9 % elastane

AGG 89 % polyamide, 11 % elastane

All VENOSAN® AES anti-embolism stockings have an inspection opening at the toes

Colour code:

Toes = indicates size

Top edge (e.g. at knee, thigh) = indicates length

**PUTTING ON YOUR STOCKINGS ...**

... for easy application, follow these instructions carefully:

1. Put your hand inside the stocking foot.
2. Hold the stocking top in your other hand, pull down turning stocking inside-out stopping at the heel.
3. Remove your hand from inside leaving stocking foot inside.
4. Stretch stocking sideways at the heel and slide your foot into the stocking foot and heel. Be sure your heel is in the center of stocking heel.
5. Grasping the top of the stocking, pull over the ankle – continuing up the leg. Adjust and straighten the stocking as you work up the leg.
6. Make sure heel and toe are properly aligned. Toes should not be cramped. The knit change at the ankle should be located in the ankle rotation area.
7. Smooth the stocking, removing wrinkles over the instep and around the ankle.

*Do not turn down the top of knee length or thigh length stockings.*

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VENOSAN® is a registered trademark of SWISSLASTIC AG ST. GALLEN, Switzerland