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Instructions for use

Contents

The set contains two liners for changing and one top stocking:

- The liners are white and have a closed toe
- The overstocking is beige and has an open toe

Together, the overstocking and the liner exert medically effective compression with:

- VENOSAN® 8002 ULCERFIT (Ccl. II) from 23–32 mmHg.
- VENOSAN® 8003 ULCERFIT (Ccl. III) from 34–46 mmHg.

What is the VENOSAN® 8000 ULCERFIT compression stocking system?

You doctor has prescribed VENOSAN® 8000 ULCERFIT for you to treat a leg ulcer (crural ulcer).

The treatment system consists of two specially knitted stockings. They complement each other in their medical effect and generate a special pressure configuration.

- The liner is normally worn day and night. This stops wound dressings from slipping
- The overstocking supports your treatment during the day by guaranteeing the defined pressure

When treating a leg ulcer (wound) your doctor integrates the stocking system into the correct phase of wound management, which always includes a suitable dressing.

Indications for the compression stocking system

- Treatment of a leg ulcer (crural ulcer) after oedema reduction
- Prevention of recurrence of chronic venous ulcers

Contraindications

Do not use VENOSAN® 8000 ULCERFIT if you have:

- Advanced peripheral arterial occlusive diseases (problems with the circulation in the femoral vessels)
- Decompensated cardiac insufficiency (myocardial insufficiency)
- Septic phlebitis (bacterial phlebitis)
- Phlegmasia coerulea dolens (acute, massive venous occlusion with blue discolouration and marked swelling of the leg)
- Material intolerance
- Sensitivity disorders of the extremity

Special caution with use is required for patients with:

- Diabetes mellitus
- Mixed ulcers

Important information

- The stocking system must be fitted individually by medically trained specialist personnel.
- The stocking system may only be used together with a suitable wound dressing for treating leg ulcers.
- If your toes become discoloured during use, or if you develop unusual pain in the lower limbs, stop wearing the stockings and please contact your doctor without delay.
- Please follow your doctor's instructions and do not change your treatment yourself.
- The manufacturer bears no liability for incorrect use or any changes you yourself make to the product.

How to use VENOSAN® 8000 ULCERFIT

- Wear the liner day and night. The particularly smooth, white material in the foot part of the liner is a donning aid that makes it easier for you to put the overstocking on.
- Put the overstocking on in the morning as soon as you get up, and take it off again in the evening when you go to bed.

How to put on both stockings

See illustrations on the fold-out flap.

- 1 Pull the liner up over your foot like a normal stocking.
- 2 Then pull the liner carefully up your leg. Make sure that the wound dressing does not slip.
- 3 Pull the overstocking over the liner over your foot like a normal stocking.
- 4 Check that it fits properly at the heel. Correct this if necessary.
- 5 Now massage the overstocking evenly upwards. The stocking should end about two fingers' breadth below your knee. Smooth out any possible wrinkles so the stocking system fits as well as possible for the optimum effect.
- 6 The cuffs of the inner and overstockings should not lie directly over each other.

Handling

- Please make sure that there are no sharp or pointed edges in your shoes. These could damage the dual stocking system.
- Sharp or torn fingernails, rough skin, sharp-edged rings, bracelets, watches, etc. can damage the sheer knit of your VENOSAN® 8000 ULCERFIT during donning and doffing. Therefore, we recommend you use our textile VENOSAN® Gloves.
- If loops or loose threads develop please do not cut them off, otherwise holes or laddering could occur.
- The stockings are resistant to skin moisture, skin care creams, skin oils and other environmental influences. However, several of these influences together can impair the useful life.
- If used and cared for properly you can wear the stockings for about six months.

Care

- Daily washing by hand or in the washing machine with the delicates programme using a commercially available detergent for delicates and a washing machine bag prolongs the useful life of the stockings.
- The liner can be washed at temperatures up to 60°C, the overstocking up to 40°C.
- Please never use fabric softeners!
- Rinse the stockings well in clear water after washing.
- Do not wring out the stockings or dry them in a tumble drier! Press the water out between two dry towels and leave the stockings to dry in the air.
- Direct heat (e.g. radiators, direct sunlight, etc) damages the stockings, as does ironing or cleaning with benzene or chemicals.

Storage

- Store the stockings at room temperature and protected against moisture and direct sunlight.

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VENOSAN® is a registered trademark of